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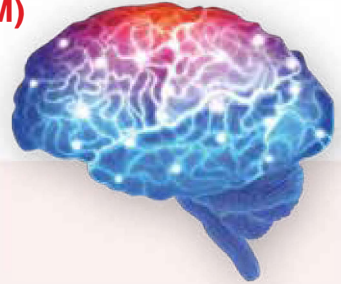
More to Life

Intended for pre-surgical embolization of brain Arteriovenous Malformation (bAVM)

Menox™ Liquid Embolic System is intended for presurgical embolization of brain Arteriovenous Malformation (bAVM) and available in 3 variants :

Menox™ LES

18 cSt viscosity,
5% EVOH

20 cSt viscosity,
6% EVOH

34 cSt viscosity,
7% EVOH

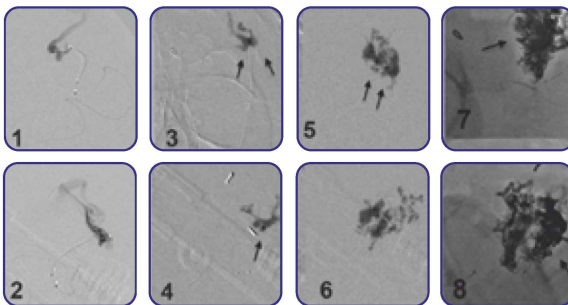
Dimethyl sulfoxide (DMSO) (1.5 ml vial)
Acts as a flushing agent.

- **1 ml DMSO compatible with luer lock syringe (3 nos.)**
 - 2 white piston syringe
 - 1 blue piston syringe

Step-by-step embolization of a ruptured left-sided parietal AVM, Spetzler-Martin grade II. 6 months follow-up (n=16)*

Study Details

The tip of the microcatheter was placed into the nidus (1 and 2), with Menox 18 successfully achieved distal and continuous nidal penetration (3-6; arrows). Note the distal extent of the Menox agent (7, 8; arrow) and the different radio-opacities of the Liquid Embolic Agent.

Study Results

Study Group (N=16)	Value
Liquid Embolic Agent	Menox 18
Total fluoroscopy time (sec.)	2460 (900-3761)
Total injection time (sec.)	325 (480-980)
Time per single injection (sec.)	21.1 (12-50.1)
Injection pause time (sec.)	1.1 (0.4-30)
Injections(no.)	11 (8-35)
Total volume of LEA per procedure (mL)	4.1 (1.2-6)

- **There were no procedural & post-procedural complications/mortality seen.**
- **Clinical success and complete obliteration were confirmed with at least 1 follow-up angiography in 10/16 patients.**

*A. Sirakov, S. Sirakov, et al; AJNR A JNeuroradiol April 2020- 14:681-86.